Revelador de placas bacterianas dentales

Un tono - Dos tonos

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Plaque disclosing solution

One Tone – Two Tone

Bacterial plaque is an invisible film, composed of bacteria that accumulate every day, especially in the interdental spaces and in the cervical region. This plaque is the main cause of dental caries and periodontal diseases, therefore its complete elimination is important.

TESTPLAC® contains innocuous dyes that temporarily stain the plate in order to allow proper brushing. The periodic use of TESTPLAC® contributes to better oral hygiene and the control of dental diseases.

Presentation form:

Green (One Tone): Home use. It dyes the bacterial plaque green. Plastic dropper bottle x 25 ml.

Yellow (One Tone): In office use. The revealing action manifests itself with the appearance of a fluorescent yellow color when it is illuminated, preferably with halogen light or LED. It has the advantage that if the plaque is not completely removed, it goes unnoticed in the mouth when the patient leaves the office. Plastic dropper bottle x 25 ml.

Two Tone: It has the particularity that it is able to distinguish the old, thicker plate from the new, recent and thin plate. It dyes blue the first and red the second. Plastic dropper bottle x 10 ml.

Instructions for use:

- 1) Wet a cotton swab with TESTPLAC® and apply it to all surfaces of the teeth.
- 2) Remove the excess by rinsing with a small amount of water.
- 3) Observe the teeth with good lighting. If you use **Two Tone**, the plaque stained blue will indicate old deposit and the new one will appear stained red. When using **One Tone**, the whole plaque will be stained green or yellow.
- 4) Make a proper brushing until the color disappears.
- 5) Apply TESTPLAC® again in order to ensure the complete elimination of the bacterial plaque.

Precautions and warnings:

- Use gloves
- Children can use TESTPLAC® under the supervision of an adult.

Composition:

TESTPLAC® Two Tone is an aqueous solution based on FD&C Green N° 3 at 1% and FD&C Red N° 3 at 1% formulated so that one does not alter the action of the other.

TESTPLAC® Yellow contains 0.75% sodium fluorescein in glycerin.

TESTPLAC® Verde contains green colorant for 0.80% food in glycerin.

Storage:

Store at temperatures between 4° and 40° C and protected from light.

To be sold to professionals and healthcare institutions only.

Authorized by the ANMAT PM 1277-22

Made in Argentina

Manufactured by:

TEDEQUIM S.R.L.

Bv. De los Polacos 6136, Los Boulevares.

X5147GGP, Córdoba, Argentina

Tel/FAX: 54-3543-448260

www.tedequim.com

Technical Director: Miryam Della Vedova – Pharmacist - License 2107